

PO BOX 180 Hamilton 3300 email: s.cotton@dynamicag.com.au Tel: (03) 5537 9446 Mob: 0447 352 321

Dr Steve Cotton B.Ag Sci (Hons), PhD (Parasitology) -Director Karen Brewis – Laboratory Operations Manager - 0429 413 780

ABN: 69 634 200 089

How to collect faecal samples for testing

BULK TESTING

1. Collect 10-20 individual faecal samples directly from freshly yarded animals (eg have been in yards < 4hrs). Select animals randomly. Collect ~5g/hd

OR

- 2. Push animals into the corner of the paddock and hold them there for 5-10min.
- 3. After 5-10min, allow animals to wander off
 - a. Collect 10-20 samples off the ground. If you can collect 30 samples, this will encompass a larger cross section of the mob/herd
 - i. Samples must be fresh. Fresh samples will be warm
 - ii. Collect an even amount of faeces from each dung patch ~5g/dung patch
- 4. Combine all the samples collected into a plastic zip-lock sandwich bag to make one BULK sample
 - a. Use 1 zip-lock bag per mob or per paddock

INDIVIDUAL TESTING

- 5. Run freshly yarded animals into a race and collect approx. 5g (1 teaspoon full) of faeces per animal
- 6. Place sample into an individual collecting container (eg plastic zip-lock bag or individual collection tray (these can be provided by Dynamic AG on request))
- 7. Label each sample with the animal's ear tag or reference number
- 8. Label the bag or tray with:
 - a. Your name
 - b. Date
 - c. Mob ID
- 9. If using a zip-lock bag, ensure you squeeze as much air out of the bag as possible and refrigerate sample(s) as soon as possible
- 10. Avoid collecting samples:
 - a. On Friday afternoon (unless they will be counted on weekend)
 - b. In hot weather (eggs can hatch quickly). Preferably collect in the morning or have an esky on hand to cool samples down once collected in the paddock
 - c. from visually weak or sick animals
- 11. Ensure you have included your lab submission form
- 12. If express posting, post earlier in the week (ie Mon/Tue) to ensure samples do not sit at the post office over the weekend

a. Wrap sample bag in newspaper to reduce smell and help keep the sample cool

- 13. Price for WECs REFER TO PRICE LIST ON THE FRONT OF THIS FORM
- 14. Results are sent via email or text message. Select the relevant option when completing your submission form